
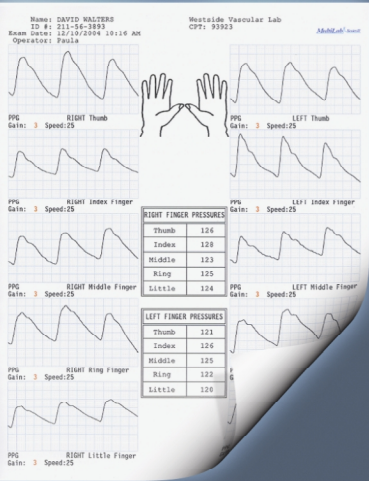
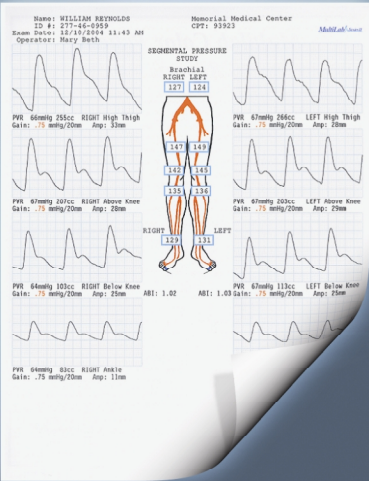
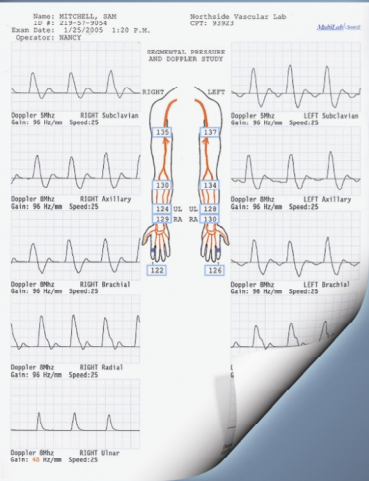


MULTILAB®

Reports

Full Size Color Samples

To Help In Your
Purchasing Decision

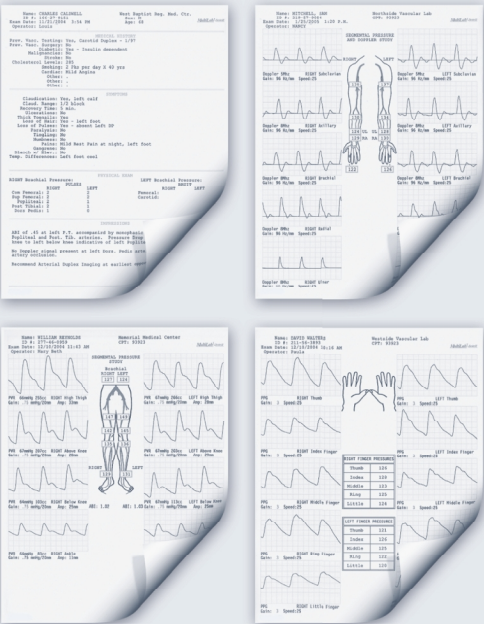

SM 11 rev E

MULTILAB®

Reports


Full Size Color Samples

To Help In Your
Purchasing Decision



Claudication: NO	Rest Pain: NO	Ulcerations: NO
Coronary Disease: YES	Tobacco: YES	Diabetes: YES
Hypertension: YES	High Cholesterol: YES	Loss of Pulses: NO
BMI >30 - Obese: NO	Family History PAD: YES	History of PAD: NO

Diagnostic ABI & TBI with
C-PVR Waveforms and Digit Waveforms

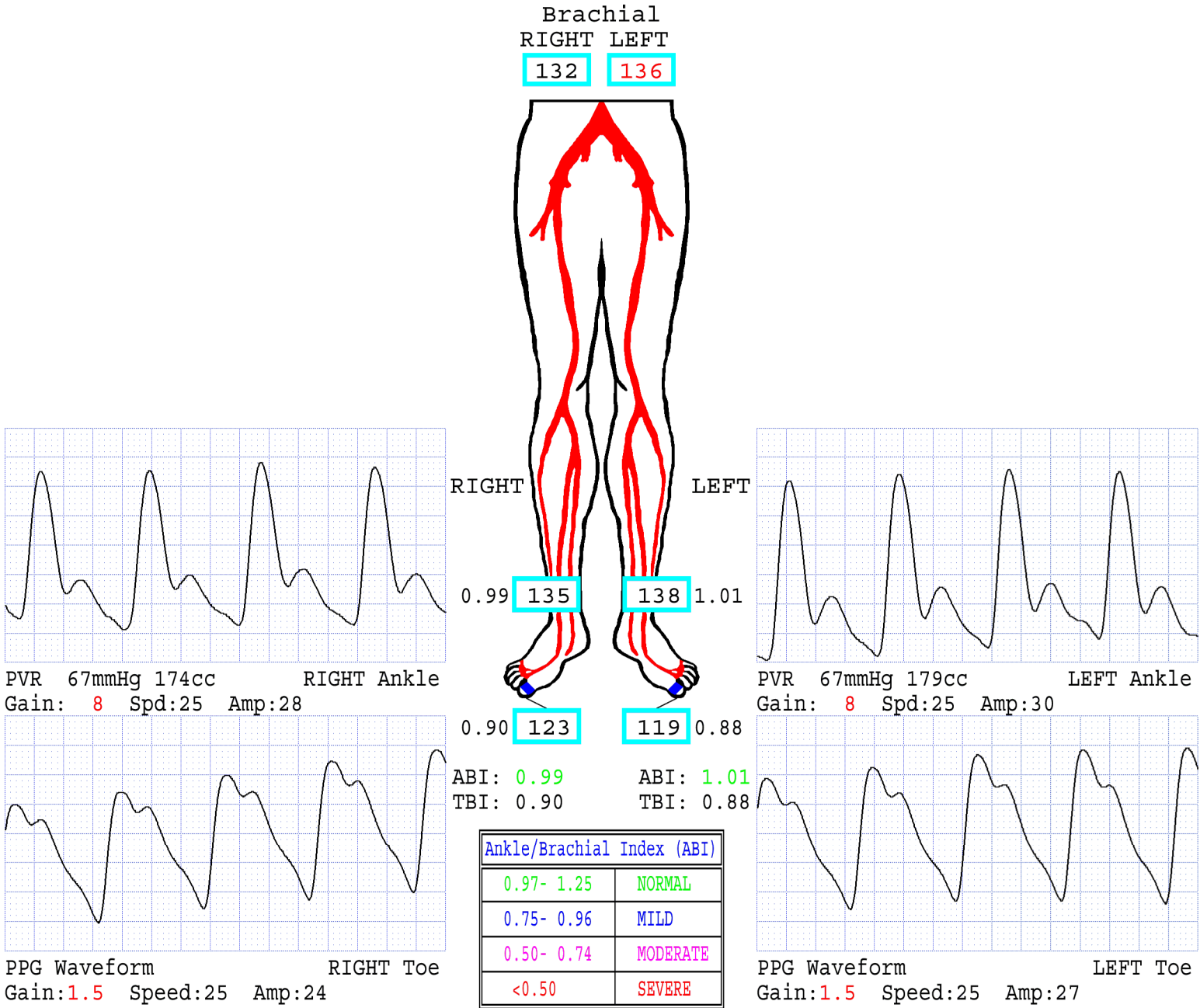

NOTES

NORMAL PAD EXAM

(*) Indices use highest brachial pressure.
(*) VESSEL CALCIFICATION MAY CAUSE FALSELY ELEVATED PRESSURES

HISTORY

Primary Indication: Rt. Calf Pain
Prev. Vasc. Surgery: Rt. Fem/Pop
Diabetic: Type II 22 Yrs
Hypertension: 15 Yrs
Stroke: No
Hyperlipidemia: Yes, Lipitor
Smoking: Quit 5 Yrs.
Cardiac: No
Other:
Other:
Other:

SYMPTOMS

Claudication: Rt Calf
Claud. Range: 100 Yards
Relieved By: Rest 5 Min.
Rest Pain: No
Thick Toenails: No
Loss of Hair: No
Loss of Pulses: No
Paralysis: No
Tingling: Feet
Numbness: Feet
Pains: No
Gangrene: No
Blanch w/ Elev.: No
Temp. Differences: No
Ulcerations: No

SEGMENTAL LIMB PRESSURES

Right	Left
Brachial: 143	Brachial: 139
High Thigh: 162 - 1.13	High Thigh: 165 - 1.15
Low Thigh: 154 - 1.08	Low Thigh: 156 - 1.09
Calf: 149 - 1.04	Calf: 149 - 1.04
Ankle (PT): 145 - 1.01	Ankle (PT): 141 - 0.99
Ankle (DP): 137 - 0.96	Ankle (DP): 135 - 0.94
Great Toe: 122 - 0.85	Great Toe: 118 - 0.83

TECH NOTES

Read By:

Name: Doe, Jane

ID #: 123-34-541k

Exam Date: 02/24/2010 08:35 AM

ACC#: :12H 355Y

University Health System


D.O.B.12/22/1945

Vascular Tech: Ann Higgin,RVT

Ref.Phys.: Dr.M.Clover

MultiLab

Series II

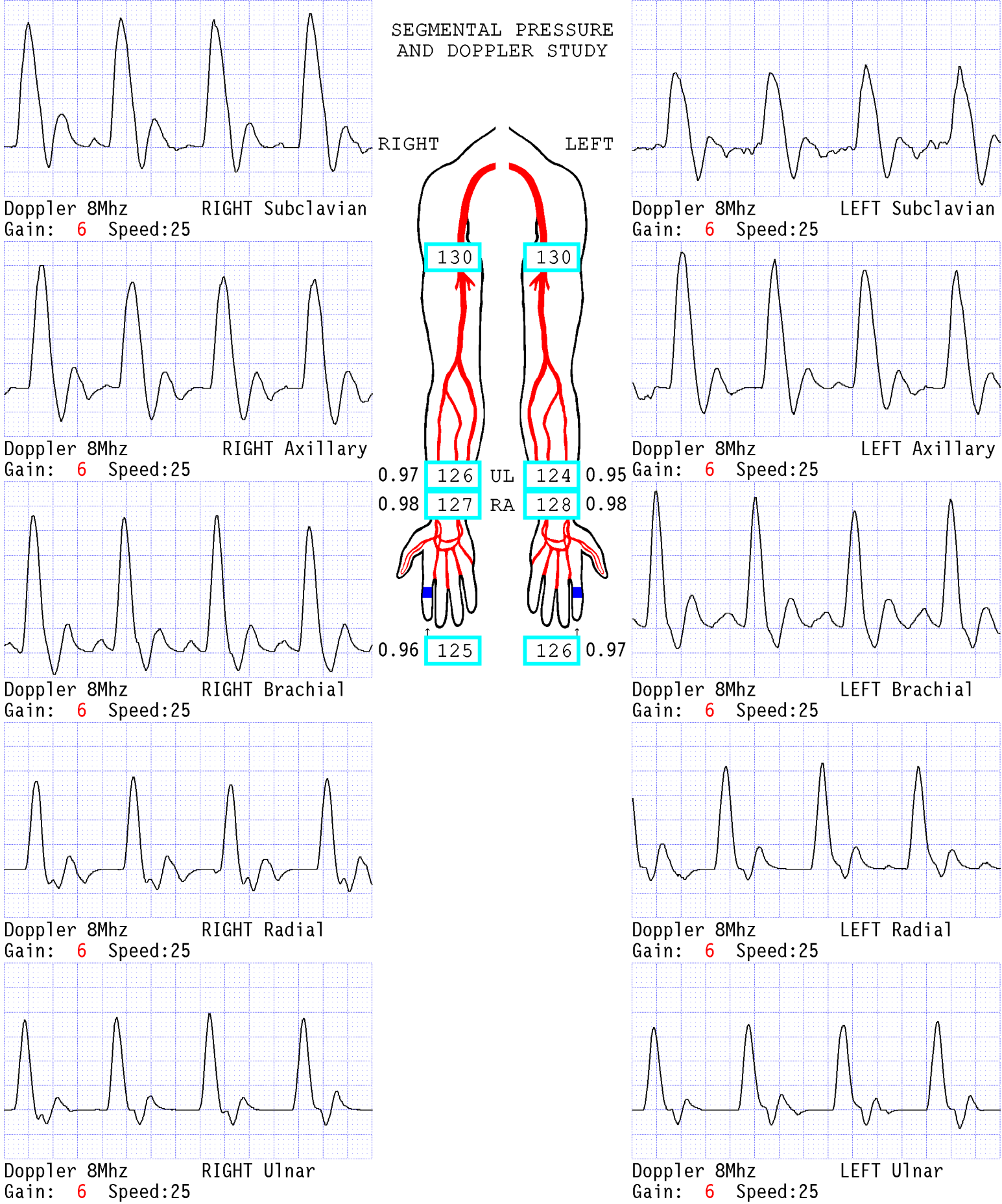

Name: Doe, Jane

ID #: 123-34-541k

Exam Date: 02/22/2010 10:59 PM

ACC#: :12H 355Y

University Health System


D.O.B.12/22/1945

Vascular Tech: Ann Higgin,RVT

Ref.Phys.: Dr.M.Clover

MultiLab

Series II

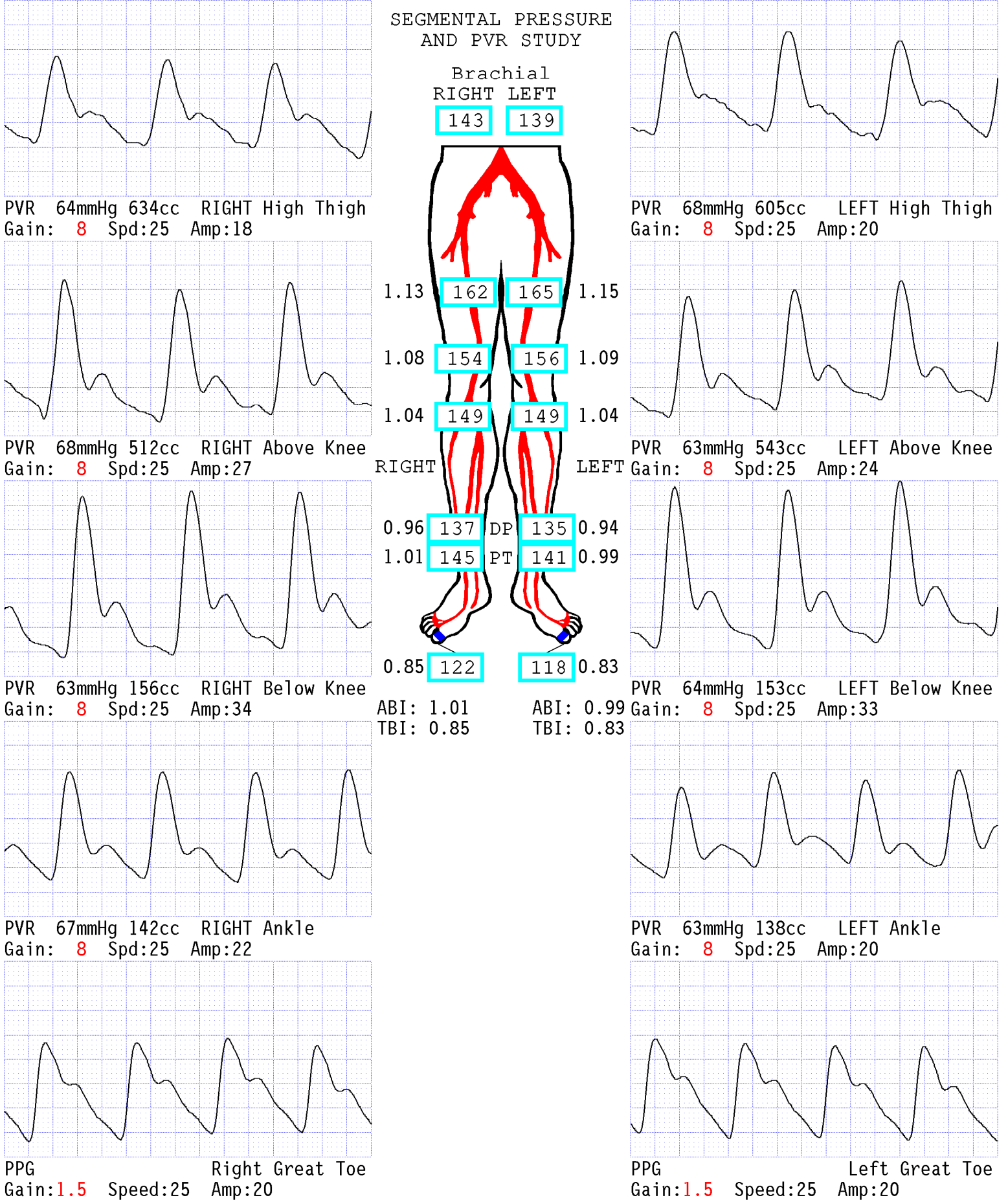

Name: DOE, JANE

ID #: 123-34-Y962

Exam Date: 02/24/2010 10:44 PM

ACC#: 12MRF-4

University Health System


D.O.B.12-22-1942

Vascular Tech: S.SANDS, RVT

Ref.Phys.:R. SHARMA

MultiLab

Series II

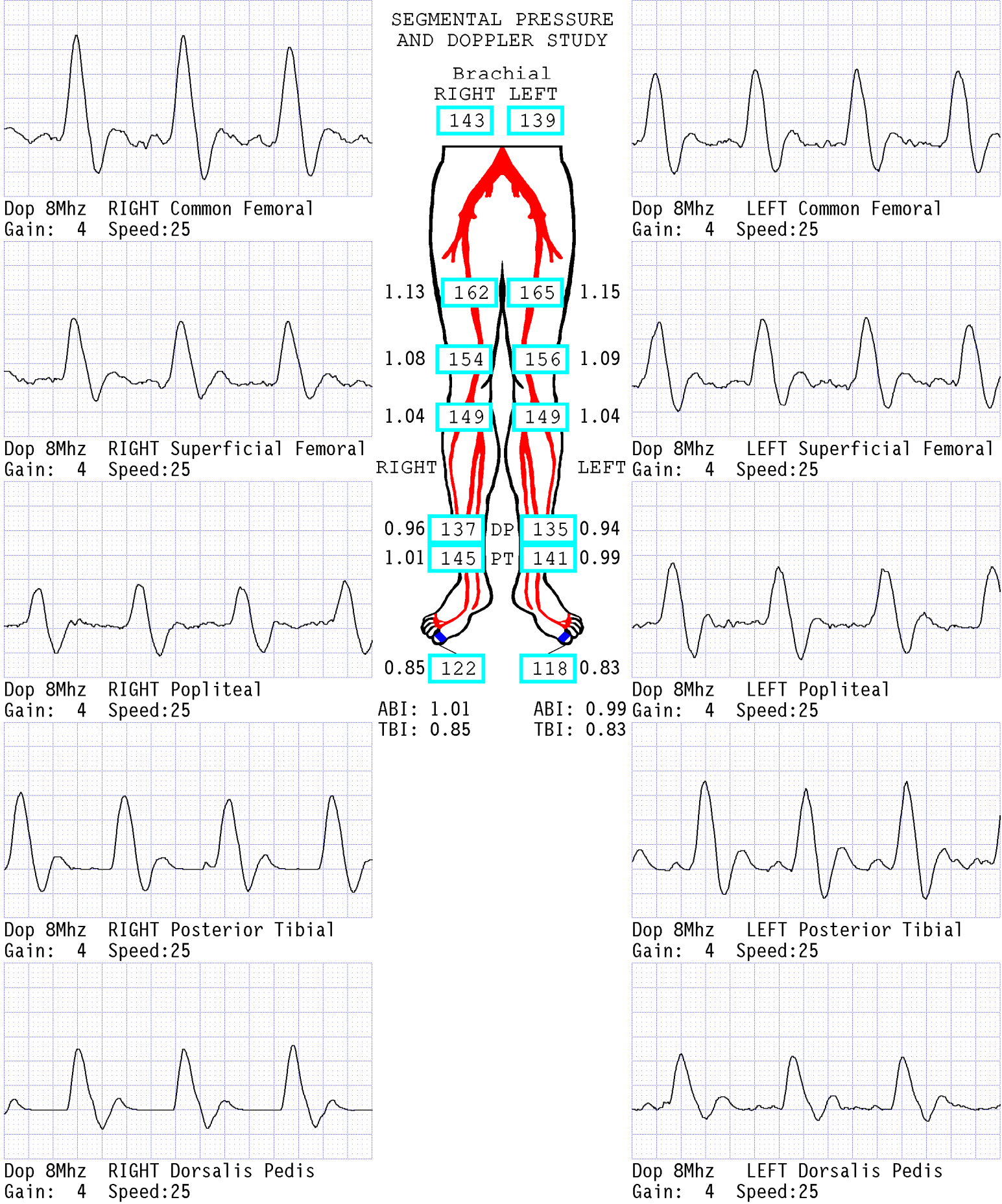

Name: Doe, Jane

ID #: 123-34-541k

Exam Date: 02/24/2010 12:11 PM

ACC#: 12H 355Y

University Health System


D.O.B.12/22/1945

Vascular Tech: Ann Higgin, RVT

Ref.Phys.: Dr.M.Clover

MultiLab

Series II

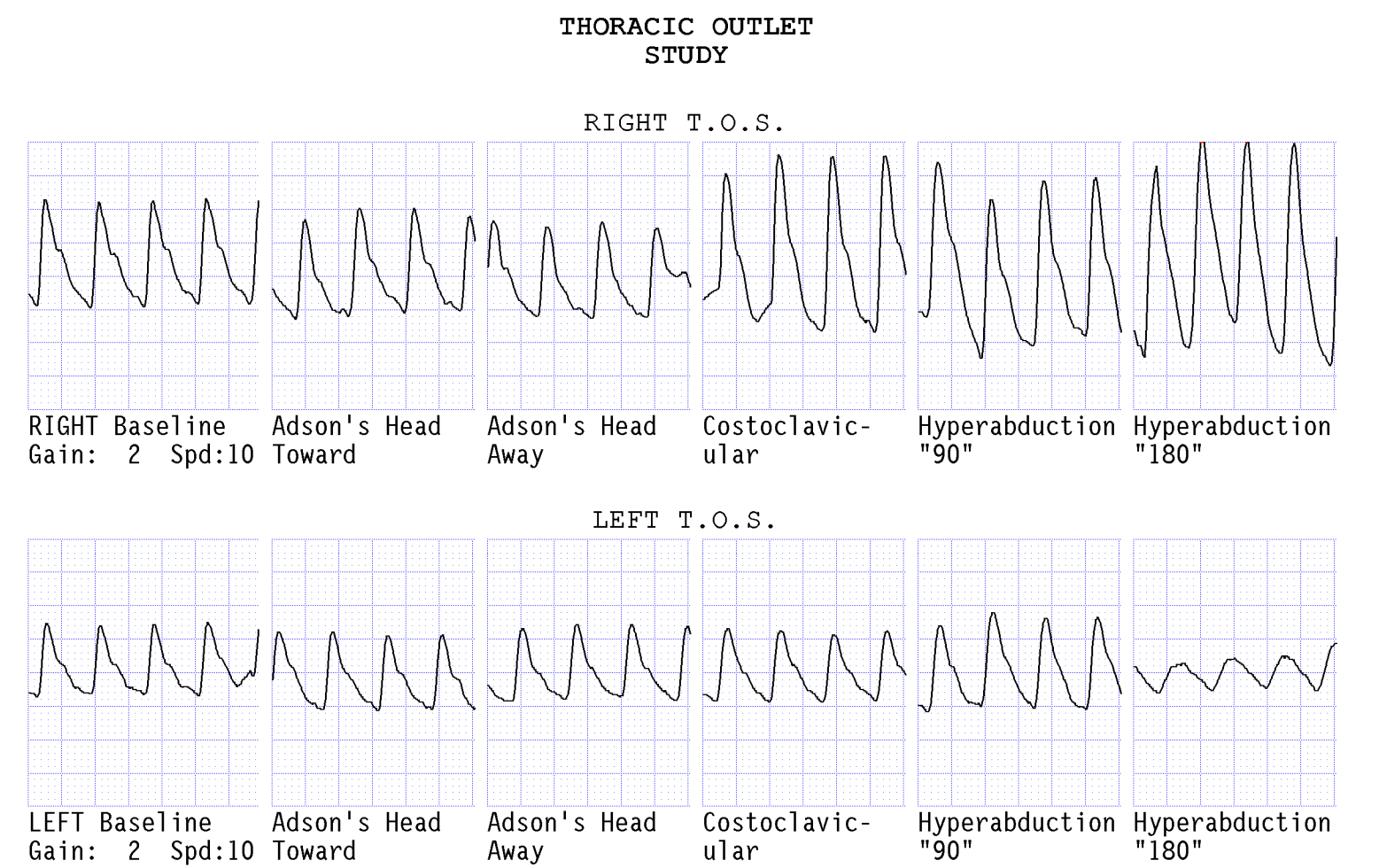

Name: Doe, Jane

ID #: 123-34-541k

Exam Date: 02/24/2010 12:17 PM

ACC#: 12H 355Y

University Health System


D.O.B.12/22/1945

Vascular Tech: Ann Higgin,RVT


Ref.Phys.: Dr.M.Clover

MultiLab

Series II

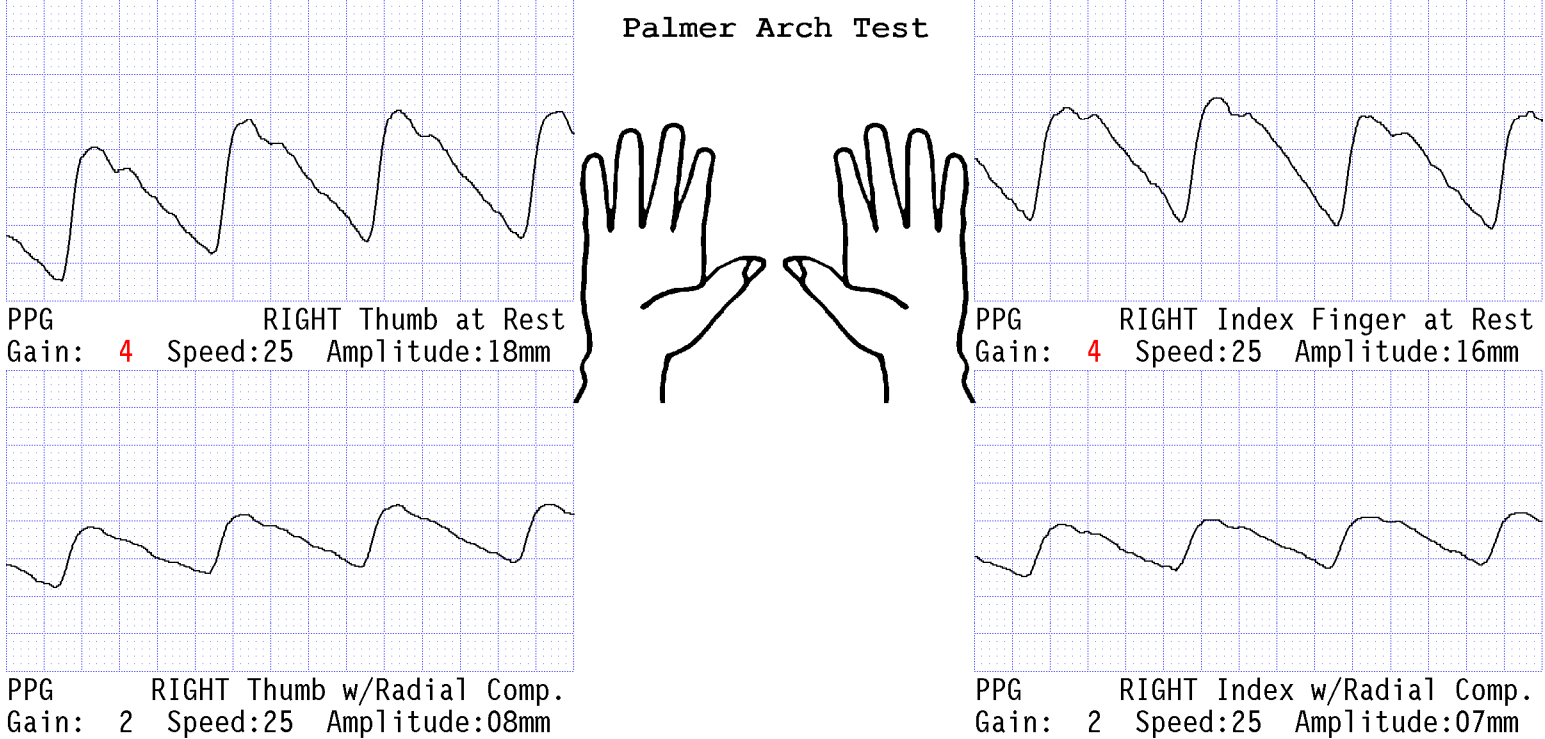

RIGHT PRESSURES & FBI			
Brachial	126		
First Dig	125	0.99	
1st w/Com	88	0.70	
2nd Rest	124	0.98	
2nd w/Com	91	0.72	

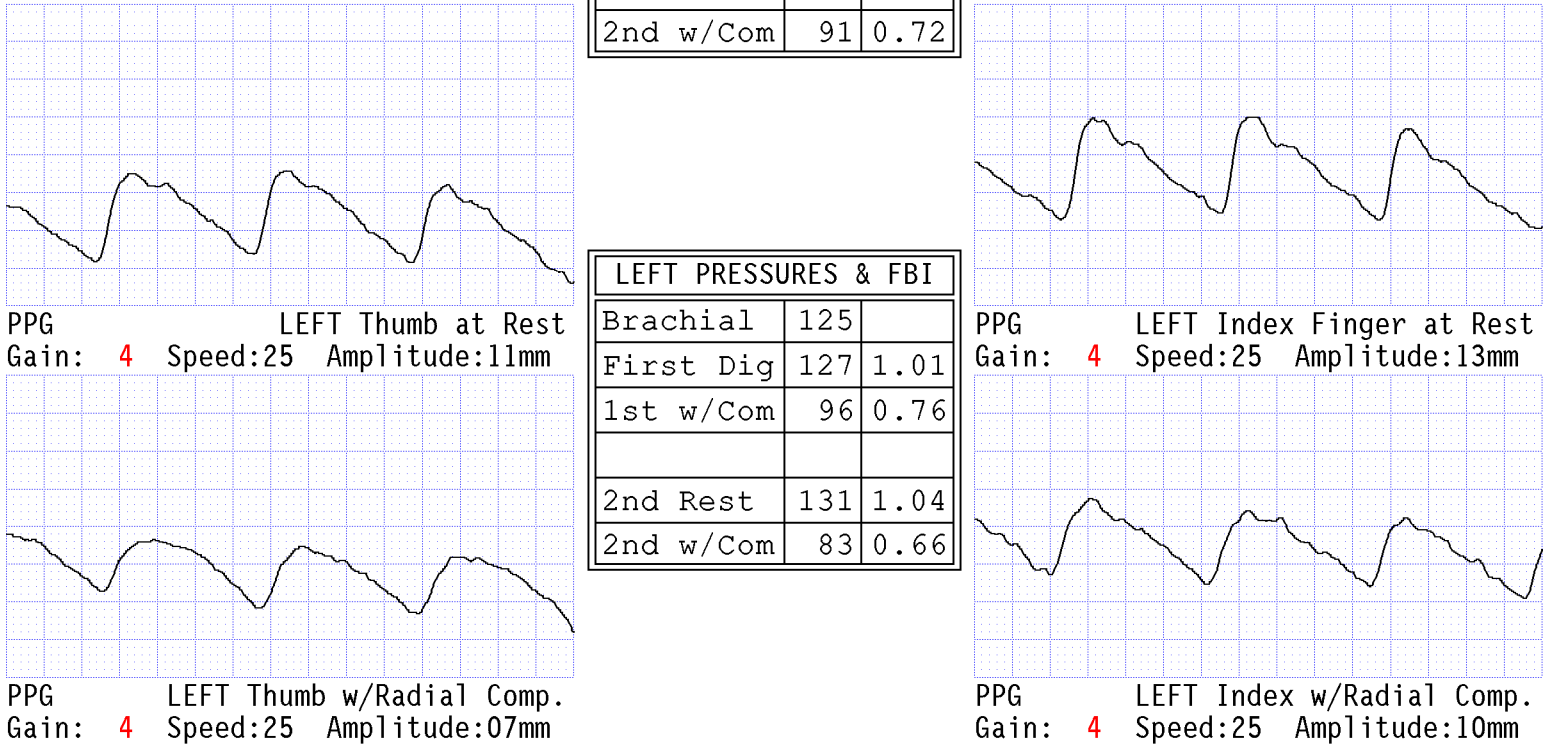

Name: Doe, Jane

ID #: 123-34-541k

Exam Date: 02/23/2010 09:16 PM

ACC#: 12H 355Y

University Health System



D.O.B.12/22/1945

Vascular Tech: Ann Higgin,RVT


Ref.Phys.: Dr.M.Clover

MultiLab


Series II

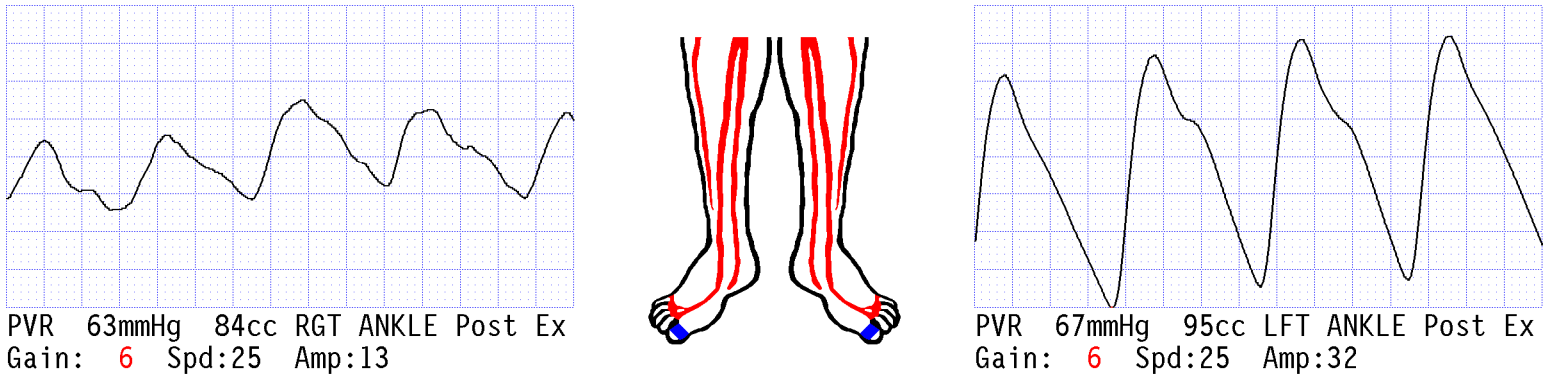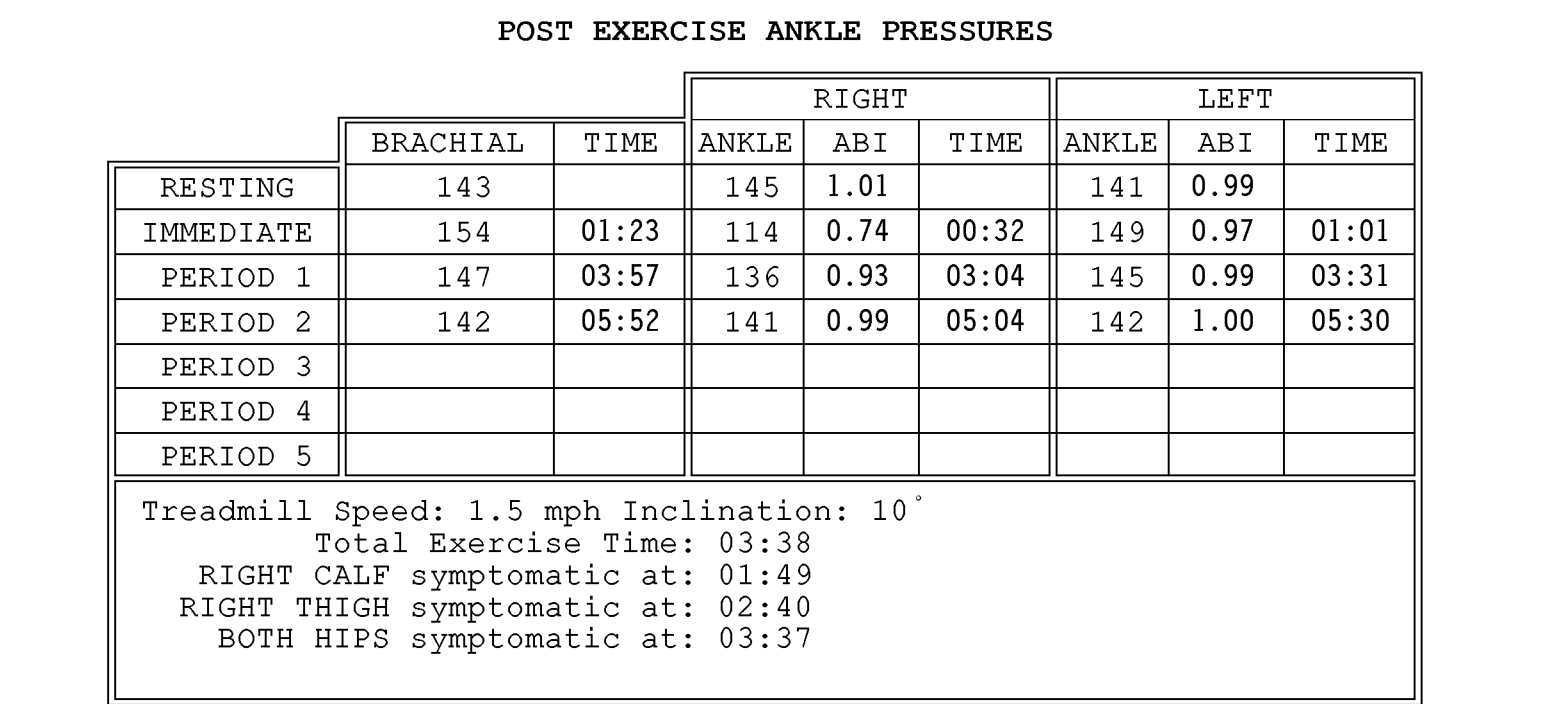


Name: Doe, Jane
ID #: 123-34-541k
Exam Date: 02/23/2010 09:59 PM
ACC#:12H 355Y

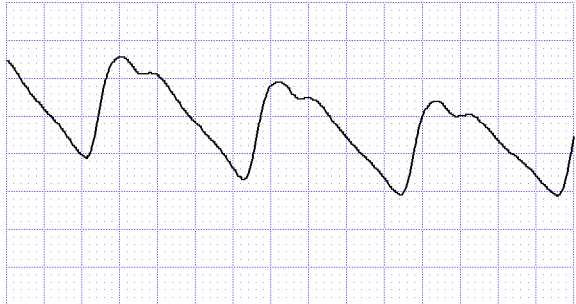


PPG Right Great Toe
Gain: 1 Speed:25 Amplitude:00mm

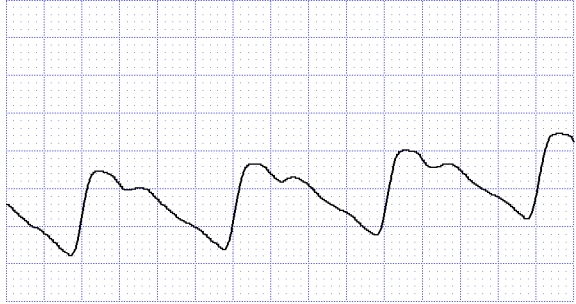


PPG RIGHT Toe 2
Gain: 1 Speed:25 Amplitude:11mm

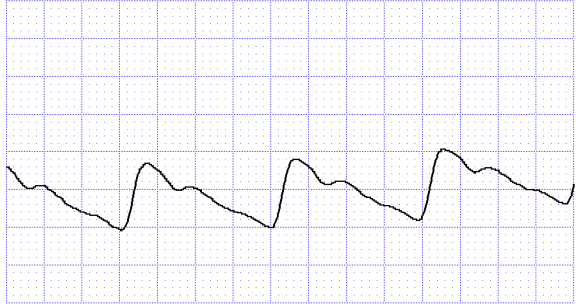


PPG RIGHT Toe 3
Gain: 1 Speed:25 Amplitude:09mm

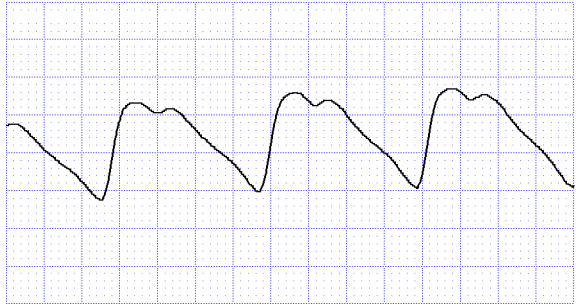

PPG RIGHT Toe 4
Gain: 1 Speed:25 Amplitude:13mm

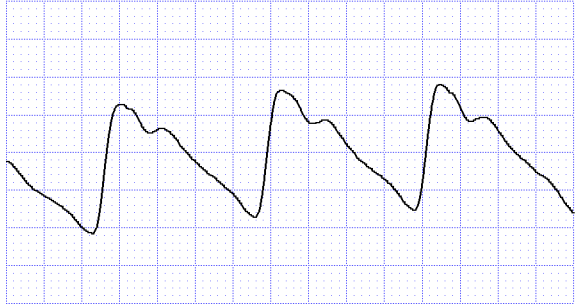

PPG RIGHT Toe 5
Gain: 1 Speed:25 Amplitude:17mm

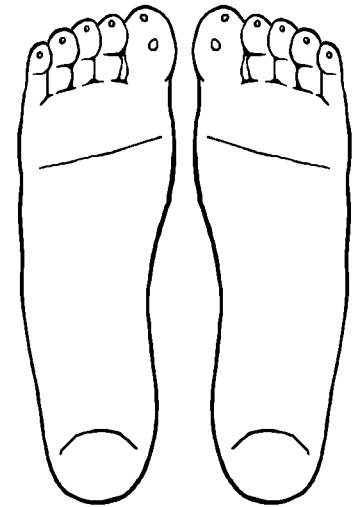


RIGHT PRESSURES & TBI			
Brachial	143		
Great Toe	122	0.85	
Toe 2	118	0.83	
Toe 3	114	0.80	
Toe 4	126	0.88	
Toe 5	112	0.78	


LEFT PRESSURES & TBI			
Brachial	139		
Great Toe	118	0.83	
Toe 2	115	0.80	
Toe 3	112	0.78	
Toe 4	118	0.83	
Toe 5	114	0.80	

University Health System
D.O.B.12/22/1945
Vascular Tech: Ann Higgin,RVT
Ref.Phys.: Dr.M.Clover


MultiLabSeriesII

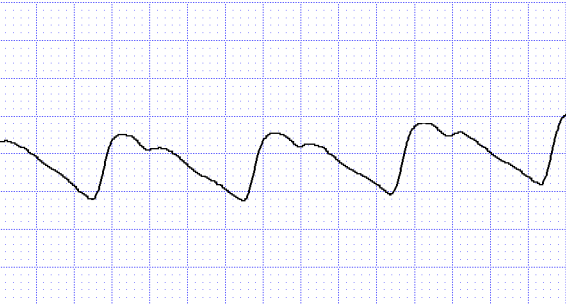


PPG Left Great Toe
Gain: 1 Speed:25 Amplitude:09mm

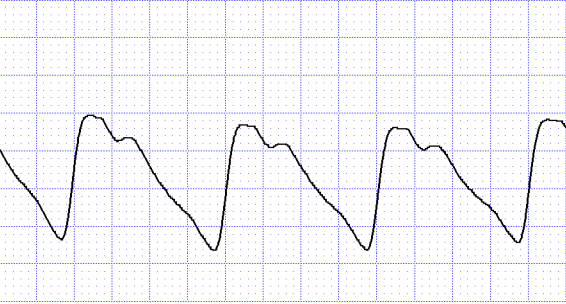


PPG LEFT Toe 2
Gain: 1 Speed:25 Amplitude:16mm

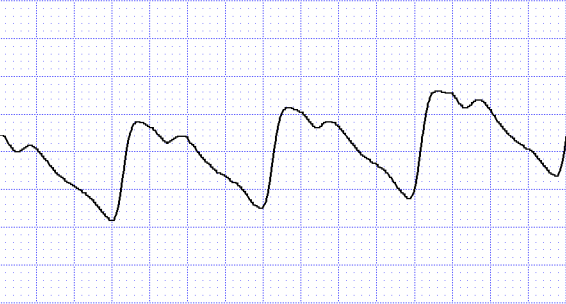

PPG LEFT Toe 3
Gain: 1 Speed:25 Amplitude:14mm

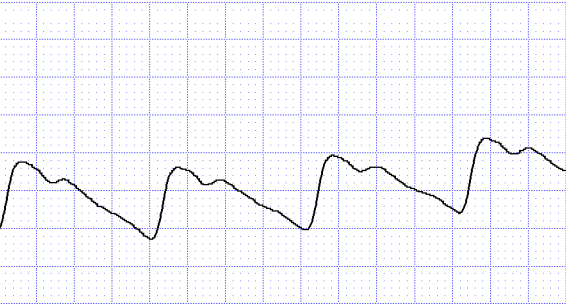

PPG LEFT Toe 4
Gain: 1 Speed:25 Amplitude:10mm

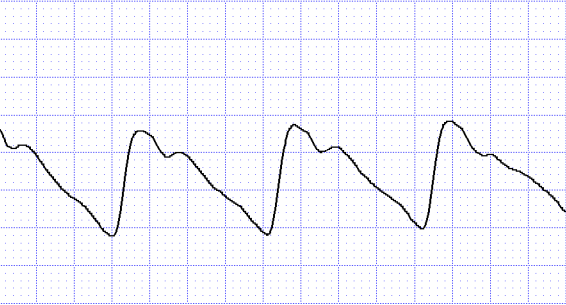


PPG LEFT Toe 5
Gain: 1 Speed:25 Amplitude:14mm

Name: DOE, JANE
ID #: 123-Y2-12WDC
Exam Date: 02/24/2010 08:27 PM
ACC#:12WDS4


University Health System
D.O.B.12/22/1956
Vascular Tech: JT
Ref.Phys.: T. SMITH

MultiLabSeriesII


RIGHT VENOUS REFILL STUDY

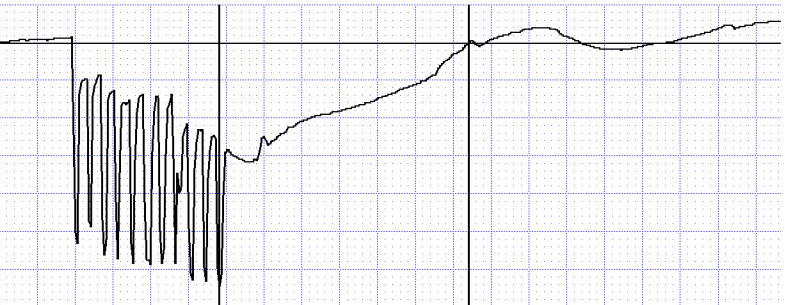

RIGHT Venous Refill VRT: 16.6 seconds
Gain: 8 Speed: 2

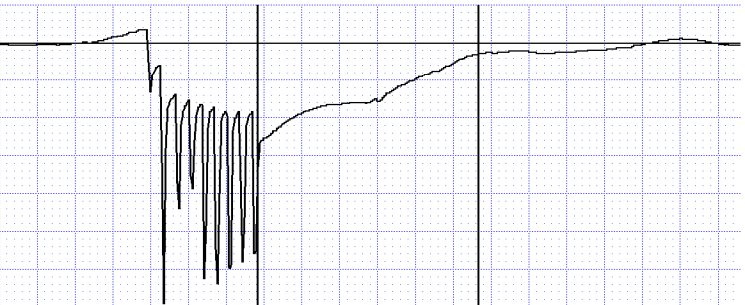

RIGHT Venous Refill w/Tourniquet Below Knee VRT: 14.7 seconds
Gain: 8 Speed: 2

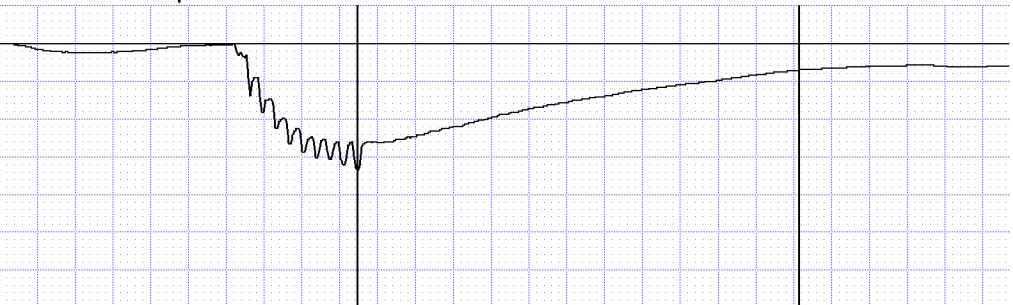

Right Venous Refill w/Tourniquet Above Knee VRT: 29.3 seconds
Gain: 8 Speed: 2

NOTES