

## Unetixs DICOM 2.0 Installer Instructions (Assumes S2 Ware 1.54 already installed)

---

Note: Before beginning the installation, contact your PACS Administrator to obtain the following values needed for step #15 below:

- Application Entity Title
  - IP Address
  - Port Number
- 1) Exit Multilab ware to desktop.
  - 2) Log off Multilab
  - 3) Log on as Administrator
  - 4) Insert floppy disk labeled "Unetixs DICOM 2.0"
  - 5) Double-click "My Computer"
  - 6) Double-click "3½ Floppy (A:)"
  - 7) Double-click "UD2Setup"  
Wait approximately one minute for the Unetixs DICOM 2.0 installer to run.
  - 8) Click "Continue"
  - 9) Click "Yes"
  - 10) Click "Continue" and wait for the installer to complete.
  - 11) Click "OK" in response to  
"Installation has been completed successfully"
  - 12) Close the "3½ Floppy (A:)" window
  - 13) Double-click the "Unetixs DICOM" icon on the desktop  
Unetixs DICOM Configuration will run.
  - 14) Click the "Modality Worklist" tab
  - 15) Set the following 4 fields to the appropriate values, examples are in italics (actual values will need to be obtained prior to installation):
    - Application Entity Title: *SIMPLEDICOM*
    - IP Address: *192.168.0.10*
    - Port Number: *105*
    - Enable (Send Query to SCP): (checked)
  - 16) Click "OK"
  - 17) Log off Administrator
  - 18) Log on Multilab  
Multilab S2 ware will autorun. If it does not, double-click the S2 icon on the desktop.
  - 19) Press "Setup"
  - 20) Press "Yes" in response to  
"Are you sure?"
  - 21) Press "System setup"
  - 22) Press "System Data"
  - 23) Advance to the "Import Control" setting (immediately follows "Export Button"). Set the following values. Settings are presented here in the order that they appear in the System Data setup (by pressing "Next" after each):
    - Import Control: c:\programs\dicommmwl
    - Import Path: c:\import\
    - Import Code: 131 (note that this is a minimum value)
    - Import Button: WRKLST (this is a suggested setting – maximum characters is 6)
    - Import Text: Perform Procedure (this is a suggested setting – maximum characters is 20)
  - 24) Press "Exit"
  - 25) Press "Exit"
  - 26) Press "Exit"  
to return to the main screen.

### Usage

- 1) On the main screen, press the Import button (you may have chosen to label it as "WRKLST" per step #23 above)
- 2) On the next screen, select a patient from the list by using the up and down arrow keys and then pressing "Select" – you will then be returned to the main screen
- 3) On the main screen, confirm that the patient info is correct as displayed above the central 3 buttons
- 4) Press "Studies" and perform studies as normal