

## **S2 1.54 Installer Instructions (for Windows 2000 and XP only)**

---

- 1) Exit Multilab ware to desktop.
- 2) Insert floppy disk labeled "S2 1.54"
- 3) Double-click "My Computer"
- 4) Double-click "3½ Floppy (A:)"
- 5) Double-click "S2-154"  
Wait approximately half a minute for the S2 1.54 installer to run.
  
- 6) Click "Continue"
- 7) Click "Yes"
- 8) Click "Continue" and wait for the installer to complete.
- 9) Click "OK" in response to  
"Installation has been completed successfully"
  
- 10) Close the "3½ Floppy (A:)" window
- 11) Double-click the S2 icon on the desktop  
Multilab S2 ware will run.
  
- 12) Press "Setup"
- 13) Press "Yes" in response to  
"Are you sure?"
  
- 14) Press "System setup"
- 15) Press "System Data"
- 16) Advance to the "Export Button" setting (immediately follows "Export Code")
  - This setting changes the text on the Export button.
  - The default setting is "Export"
  - Maximum characters is 6
  - Examples: DICOM, PACS, TOPACS, etc.
  
- 17) Press "Exit"
- 18) Press "Exit"
- 19) Press "Exit"  
to return to the main screen.